

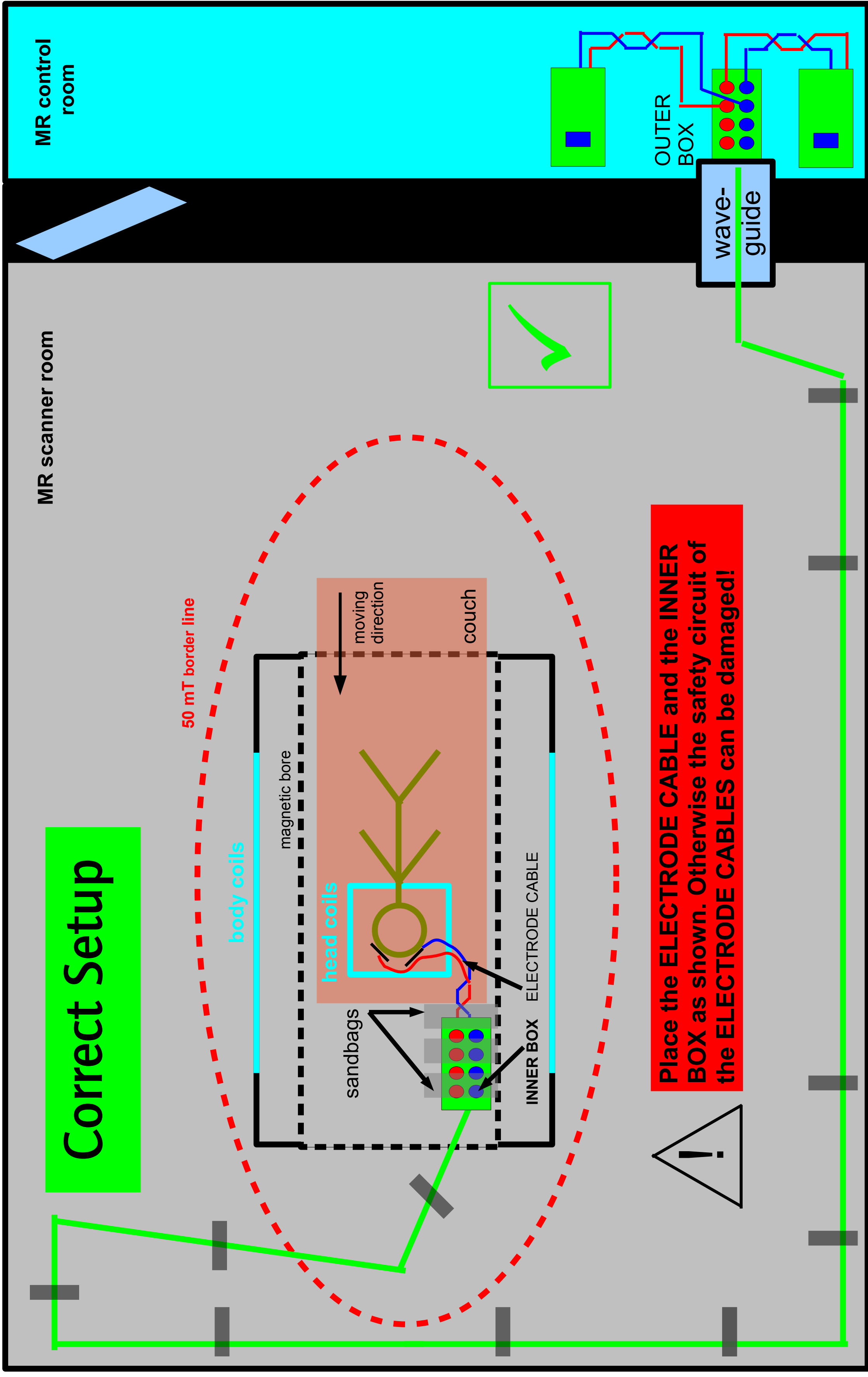


DC-STIMULATOR MR – Application Note

Dual-channel transcranial electrical stimulation (tES) during functional MRI

– Recommendations –

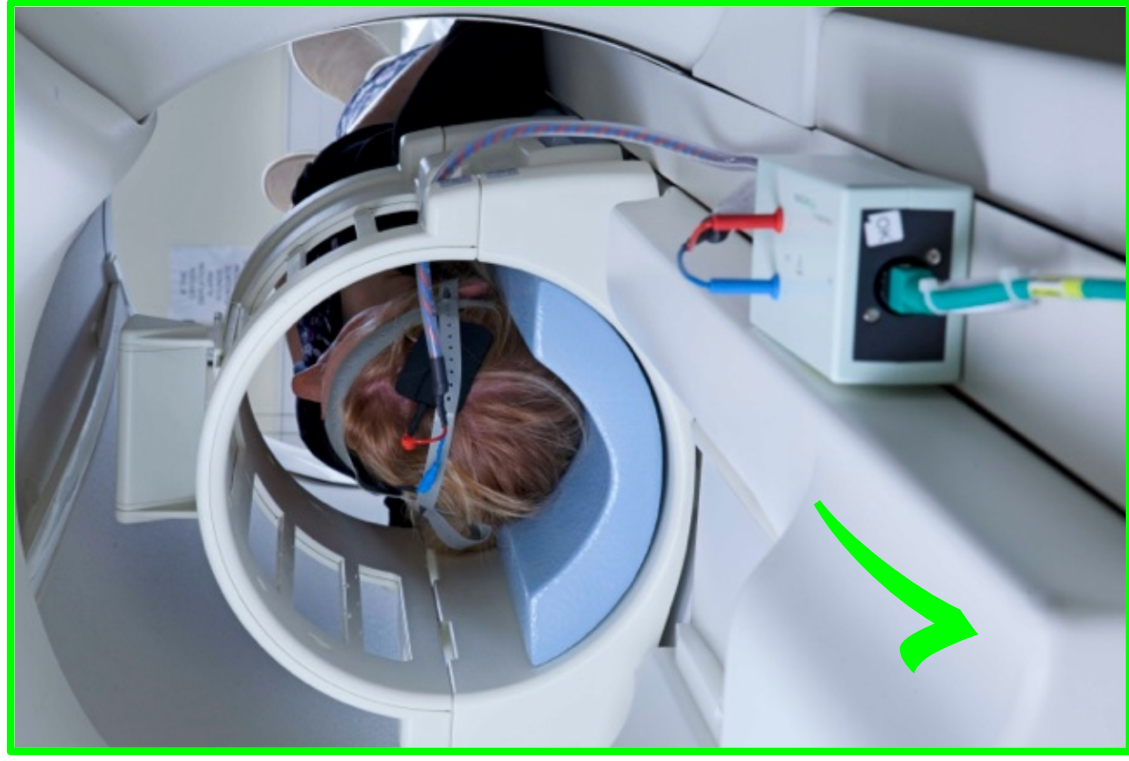


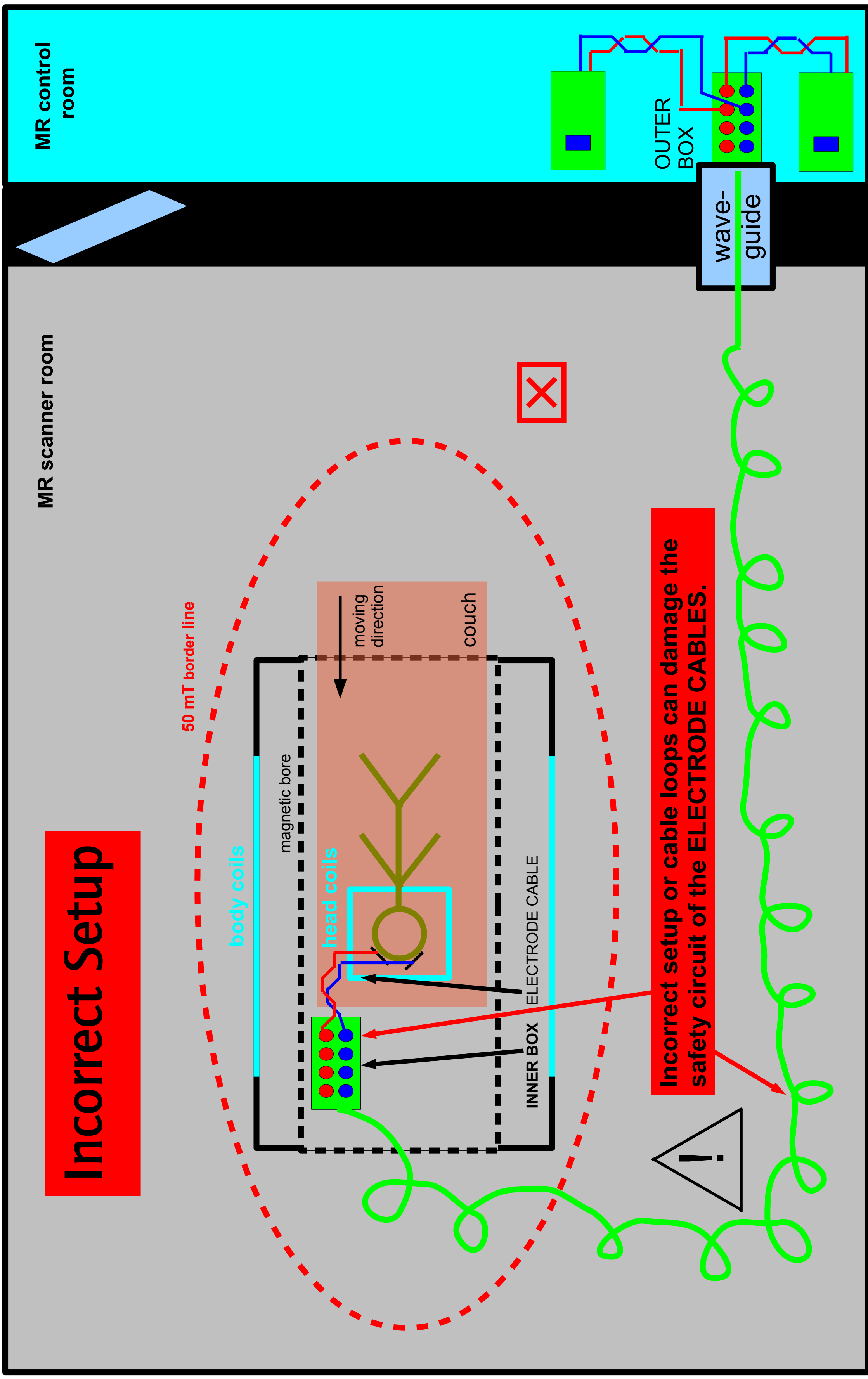

Incorrect Setup




Correct Setup

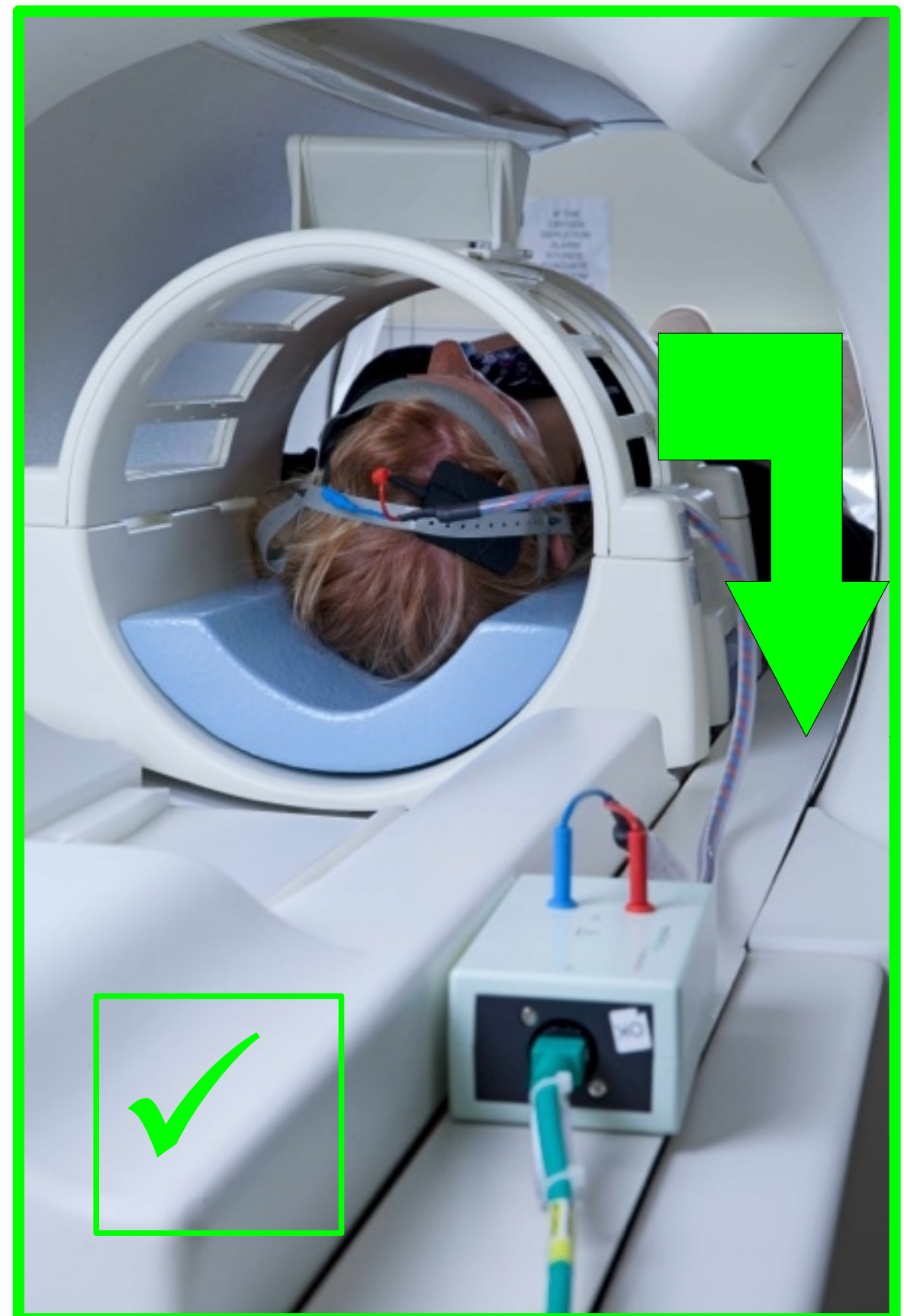



- The cable outlet MUST be on placed correctly. Otherwise the safety circuit of the electrode cable can be damaged.
- The DC-STIMULATOR MR was designed for fMRI sequences (EPI). We recommend to remove the ELECTRODE CABLE from the patient during anatomical sequences. Rubber electrodes can be left on patient's head.



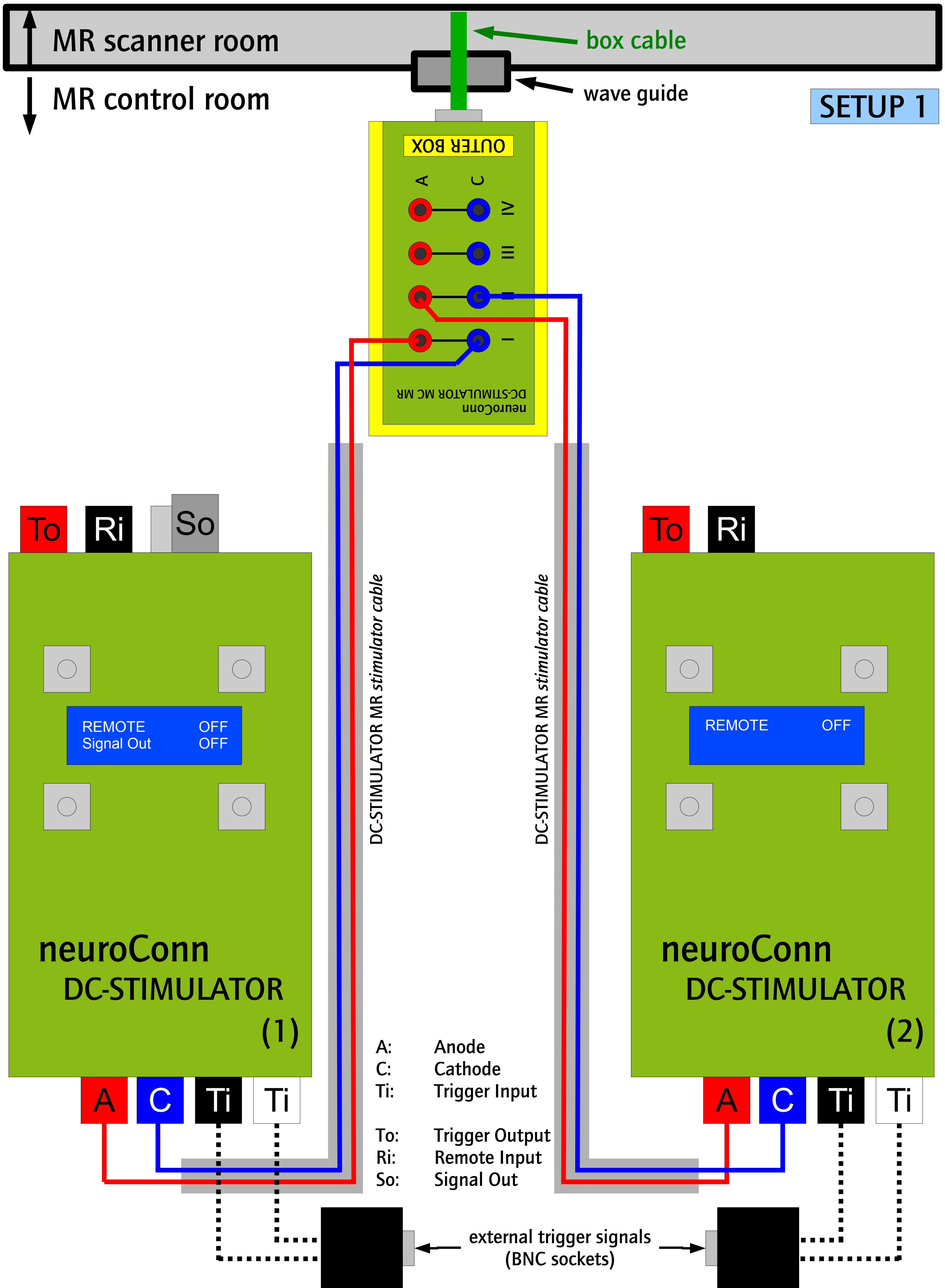


Setup 1	
waveforms	waveforms from DC-Stimulator (tDCS, sinus, sinus (hw), sinus (w), pulse, noise, noise LF, noise HF)
channel waveforms:	different or common waveforms for channel I and channel II
operating modes:	DC-Stimulator 1 (left): REMOTE OFF, Signal Out: OFF DC-Stimulator 2 (right): REMOTE OFF
start trigger:	usable, common or different trigger events for DC-Stimulator 1 and 2 trigger modes: disabled, single or repetitive
impedance check:	Channel I: double check (voltage and current), automatic stop if impedance exceeds limits, short acoustic warning, <u>SAFE-STOP</u> Channel II: double check (voltage and current), automatic stop if impedance exceeds limits, short acoustic warning, <u>SAFE-STOP</u>
clock drift:	existing
transfer gain:	—
phase delay:	—
stimulation time:	approximately 8 hours

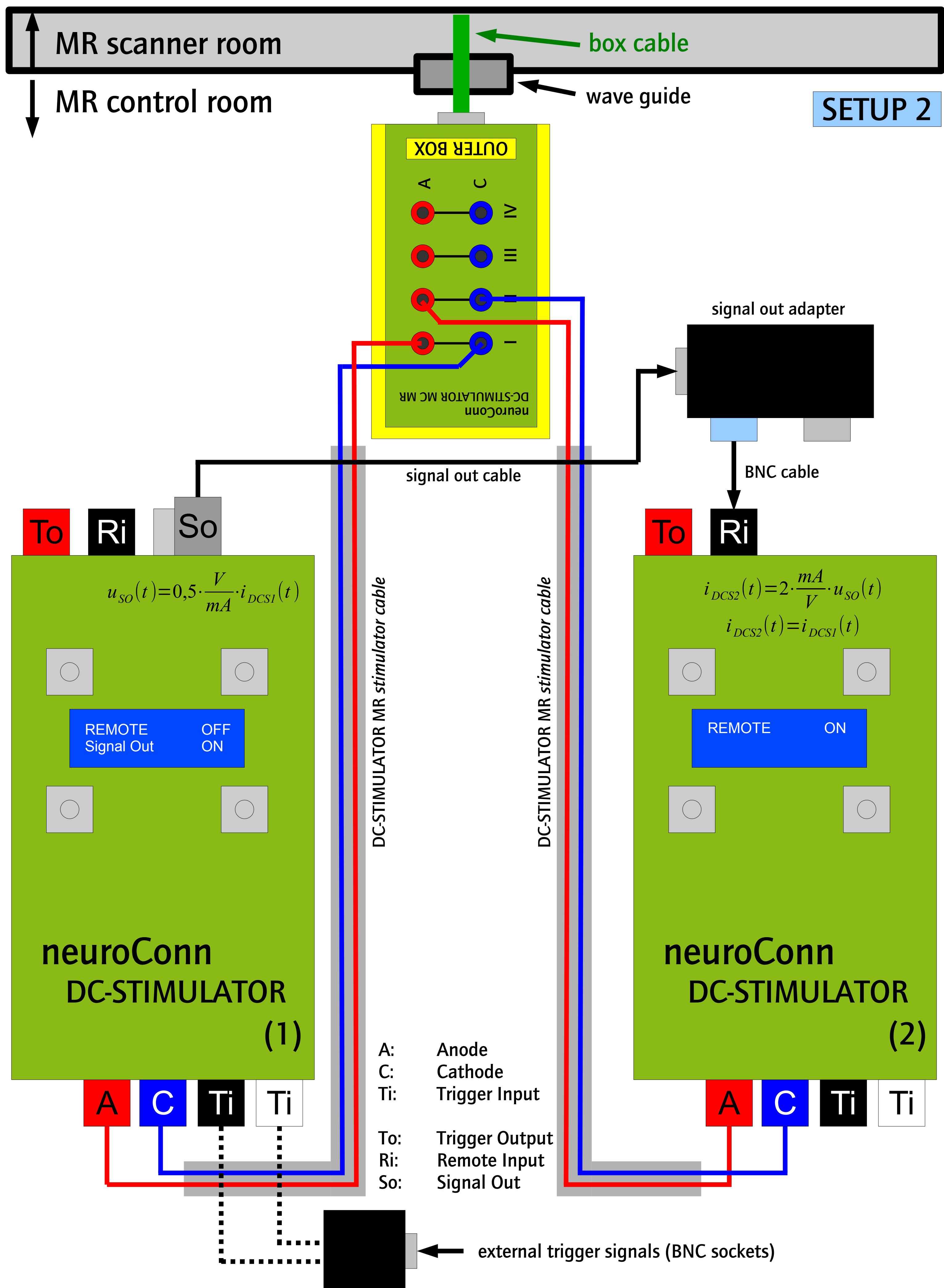



Setup 2	
waveforms	waveforms from DC-Stimulator (tDCS, sinus, sinus (hw), sinus (w), pulse, noise, noise LF, noise HF)
channel waveforms:	common waveform for channel I and II
operating modes:	DC-Stimulator 1 (left): REMOTE OFF, Signal Out: ON DC-Stimulator 2 (right): REMOTE ON
start trigger:	usable, trigger events only for DC-Stimulator 1 trigger modes: disabled, single or repetitive
impedance check:	<p>Channel I: double check (voltage and current), automatic stop if impedance exceeds limits, short acoustic warning, <u>SAFE-STOP</u></p> <p>Channel II: simple overload check (voltage), permanent acoustic warning, no automatic stop if impedance exceeds limits, <u>no SAFE-STOP</u></p> <div>  <p><u>The user is responsible for stimulation stop.</u></p> </div>
clock drift:	NO
transfer gain:	$i_{DCS1}(t) = i_{DCS2}(t)$
phase delay:	small between channel I and channel II
stimulation time:	approximately 6 – 8 hours

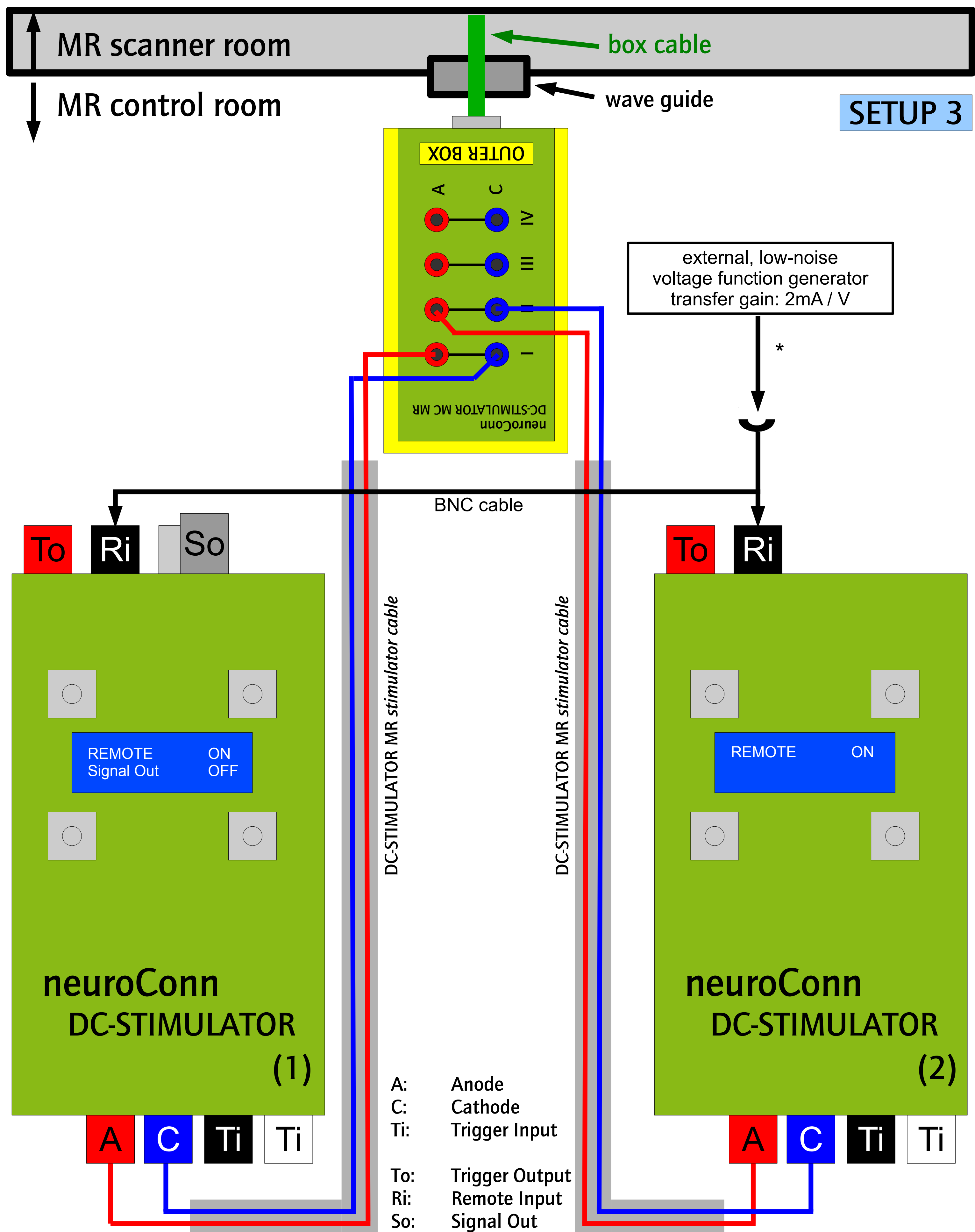




* If necessary, use the neuroConn's DC-STIMULATOR filter cable (BNC) to increase signal-to-noise-ratio.



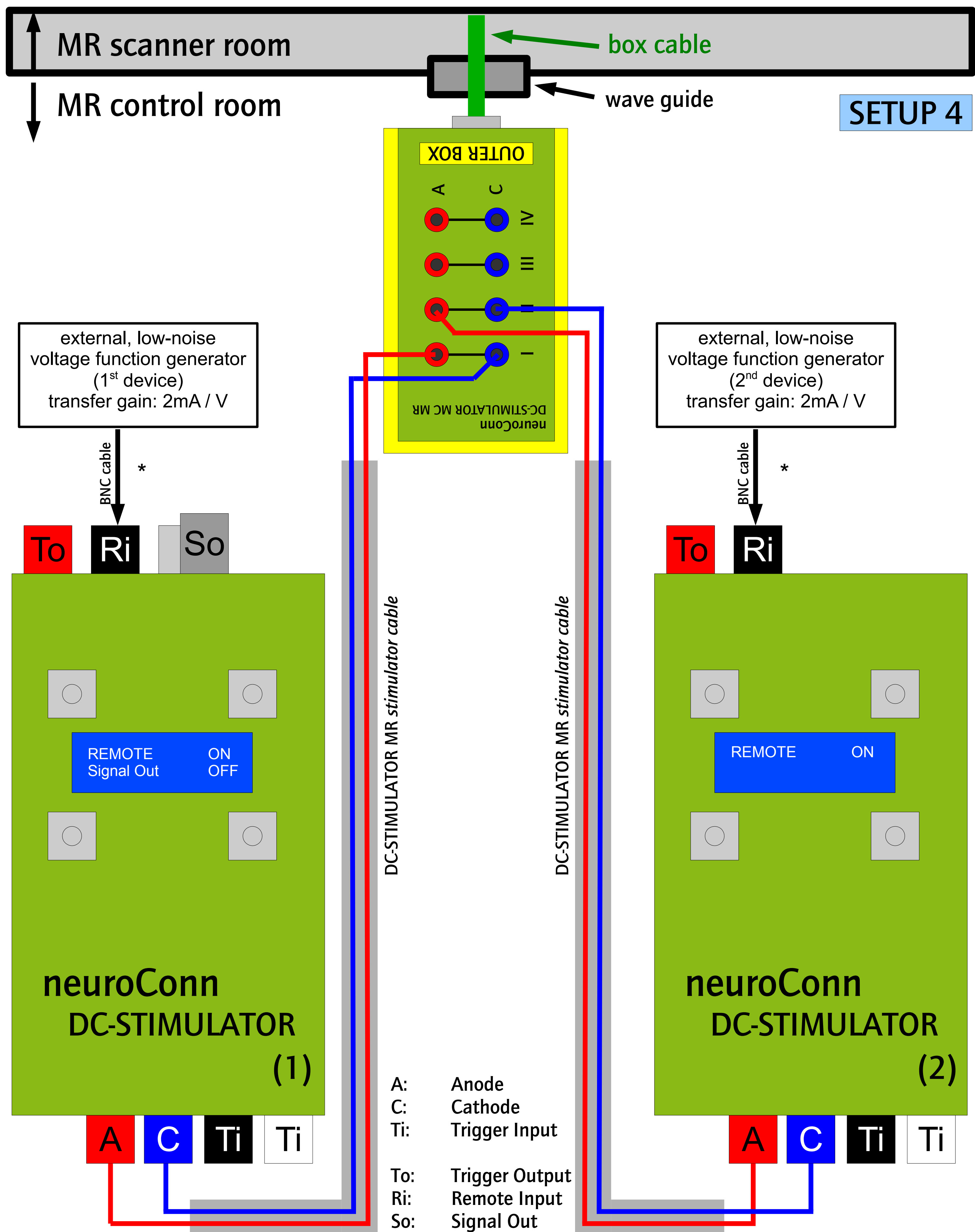


Setup 3	
waveforms	external waveform from low-noise voltage generator
channel waveforms:	common waveform for channel I and II
operating modes:	DC-Stimulator 1 (left): REMOTE ON, Signal Out: OFF DC-Stimulator 2 (right): REMOTE ON
start trigger:	not usable <u>start trigger should be connected to external voltage generator</u>
impedance check:	<div> Channel I: simple overload check (voltage), permanent acoustic warning, no automatic stop if impedance exceeds limits, <u>no SAFE-STOP</u> <div>  <div> <u>The user is responsible for stimulation stop.</u> </div> </div> </div> <div> Channel II: simple overload check (voltage), permanent acoustic warning, no automatic stop if impedance exceeds limits, <u>no SAFE-STOP</u> <div>  <div> <u>The user is responsible for stimulation stop.</u> </div> </div> </div>
clock drift:	NO
transfer gain:	2 mA / V
phase delay:	NO
stimulation time:	approximately 6 – 8 hours



* If necessary, use the neuroConn's DC-STIMULATOR filter cable (BNC) to increase signal-to-noise-ratio.



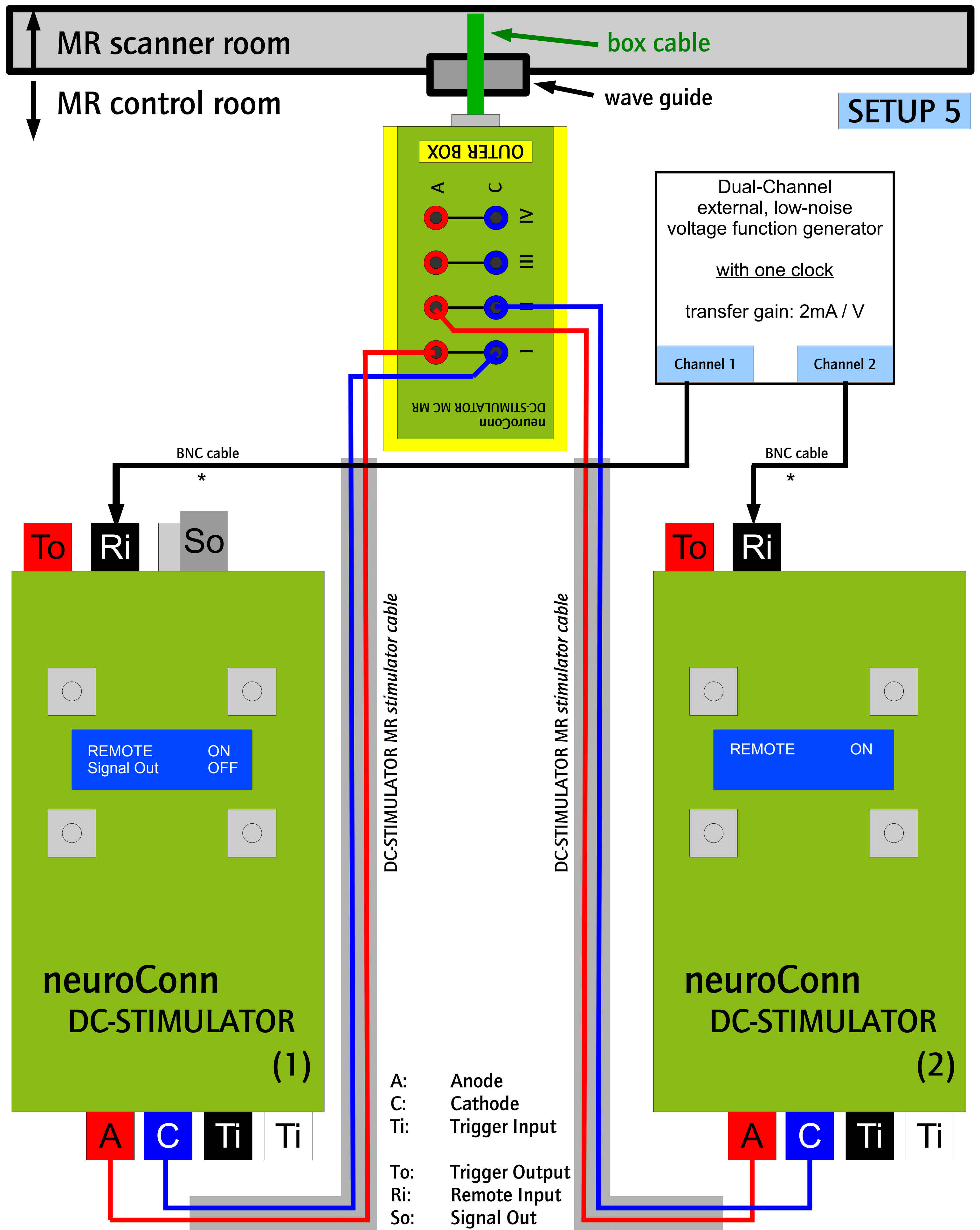



Setup 4	
waveforms	external waveform from low-noise voltage generator
channel waveforms:	common or different waveforms for channel I and II
operating modes:	DC-Stimulator 1 (left): REMOTE ON, Signal Out: OFF DC-Stimulator 2 (right): REMOTE ON
start trigger:	not usable <u>start trigger should be connected to external voltage generator</u>
impedance check:	<div> Channel I: simple overload check (voltage), acoustic warning, no automatic stop if impedance exceeds limits, <u>no SAFE-STOP</u> <div>  <div> <div>The user is responsible for stimulation stop.</div> </div> </div> </div> <div> Channel II: simple overload check (voltage), acoustic warning, no automatic stop if impedance exceeds limits, <u>no SAFE-STOP</u> <div>  <div> <div>The user is responsible for stimulation stop.</div> </div> </div> </div>
clock drift:	existing with two waveform generators, not-existing with dual-channel waveform generators
transfer gain:	2 mA / V
phase delay:	depending on waveform generators
stimulation time:	approximately 6 – 8 hours





* If necessary, use the neuroConn's DC-STIMULATOR filter cable (BNC) to increase signal-to-noise-ratio.



Setup 5	
waveforms	external waveform from low-noise voltage generator
channel waveforms:	common or different waveforms for channel I and II
operating modes:	DC-Stimulator 1 (left): REMOTE ON, Signal Out: OFF DC-Stimulator 2 (right): REMOTE ON
start trigger:	not usable <u>start trigger should be connected to external voltage generator</u>
impedance check:	<div> <div>Channel I: simple overload check (voltage), acoustic warning, no automatic stop if impedance exceeds limits, <u>no SAFE-STOP</u></div> <div>  <div> <div>The user is responsible for stimulation stop.</div> </div> </div> <div>Channel II: simple overload check (voltage), acoustic warning, no automatic stop if impedance exceeds limits, <u>no SAFE-STOP</u></div> <div>  <div> <div>The user is responsible for stimulation stop.</div> </div> </div> </div>
clock drift:	<u>not-existing with dual-channel waveform generators (derived from one clock)</u>
transfer gain:	2 mA / V
phase delay:	depending on waveform generators
stimulation time:	approximately 6 – 8 hours

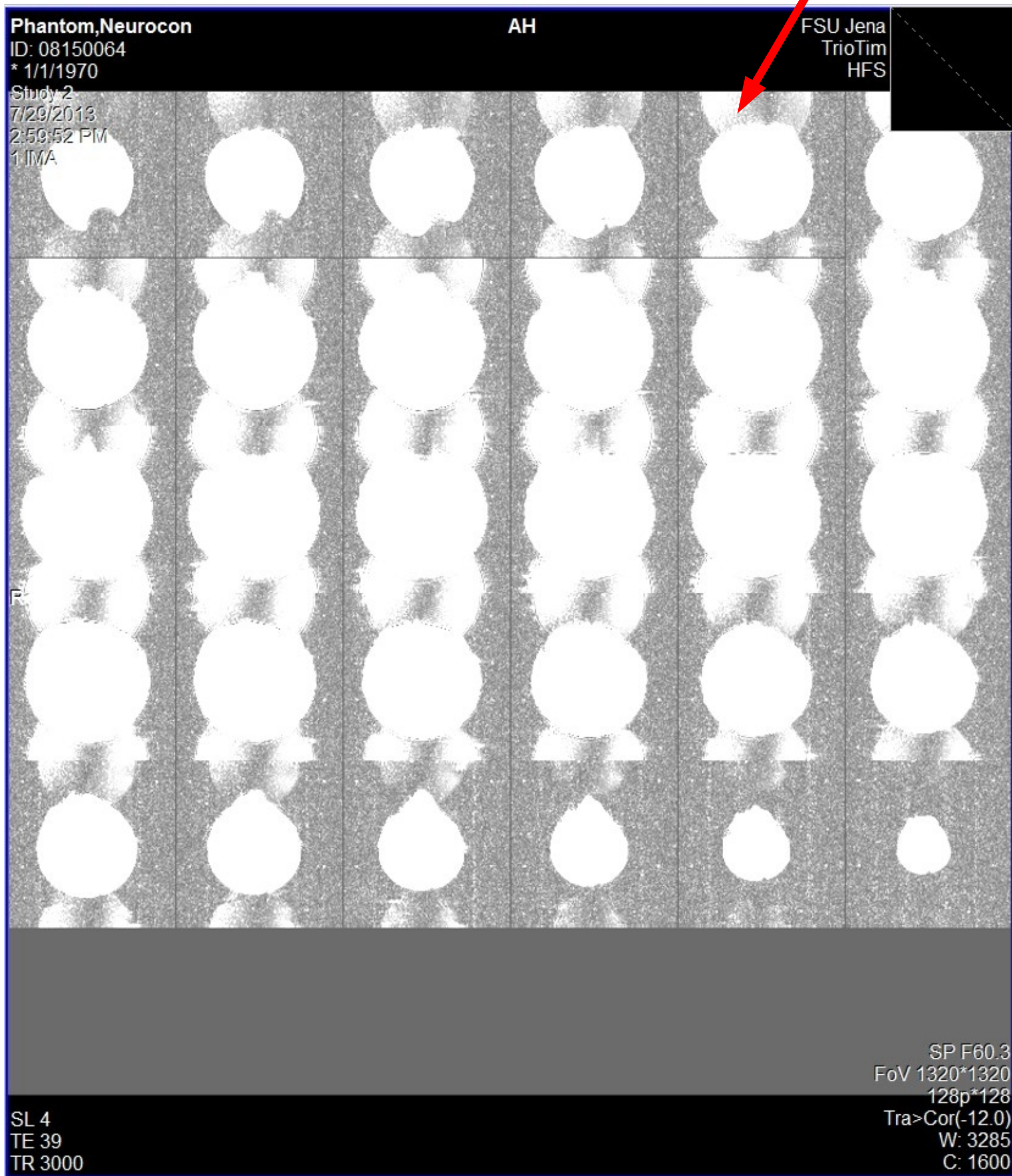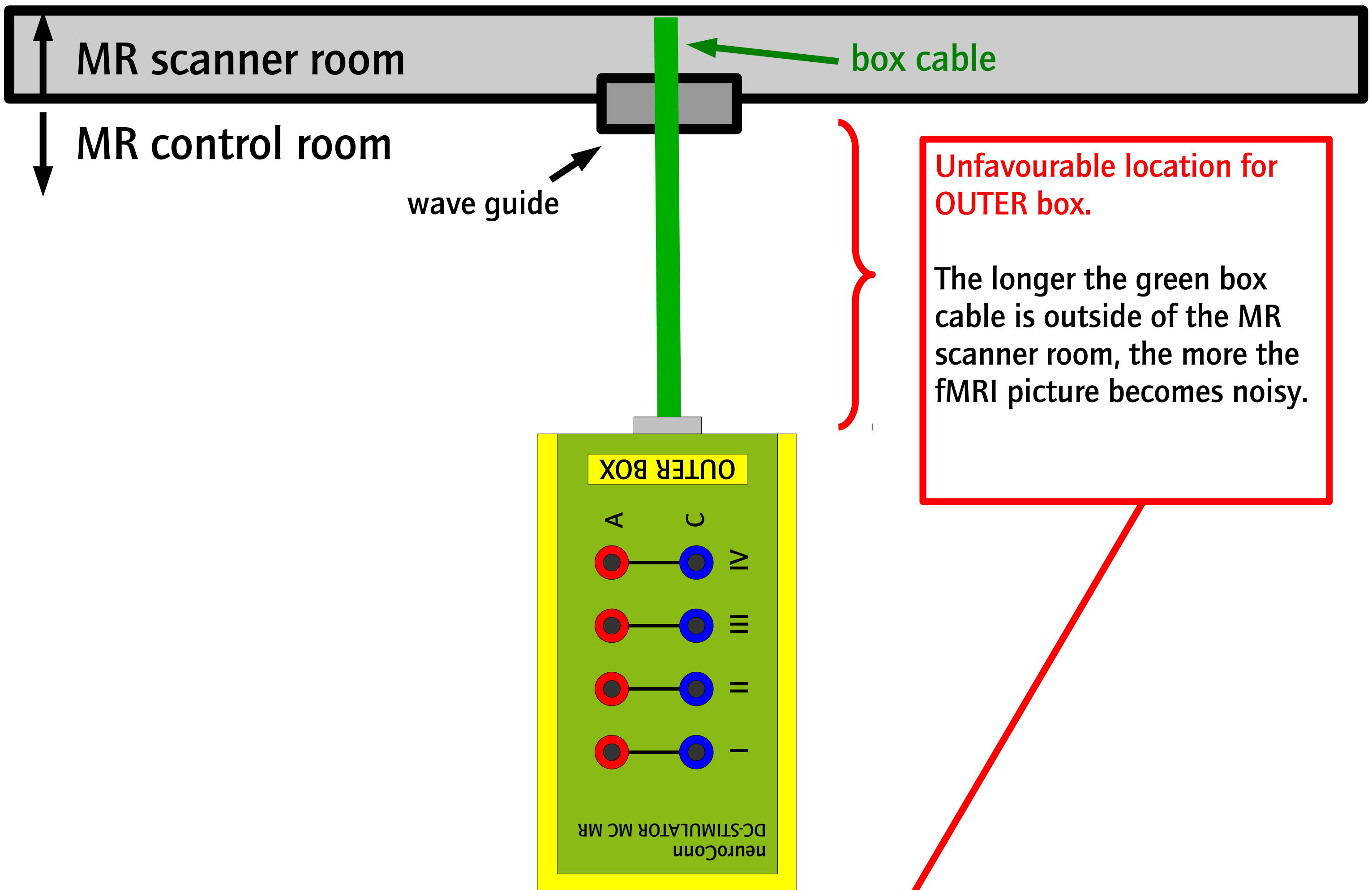

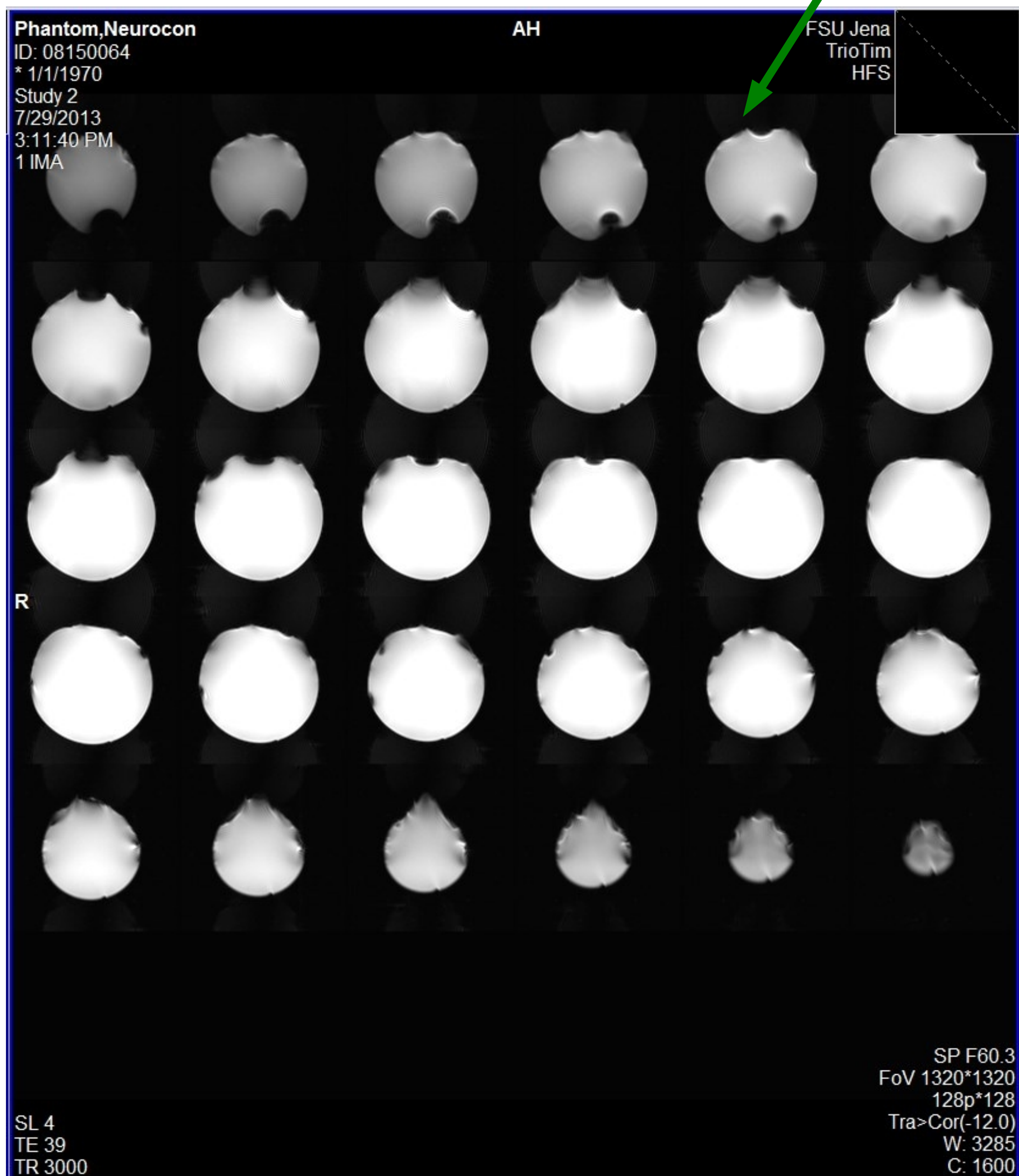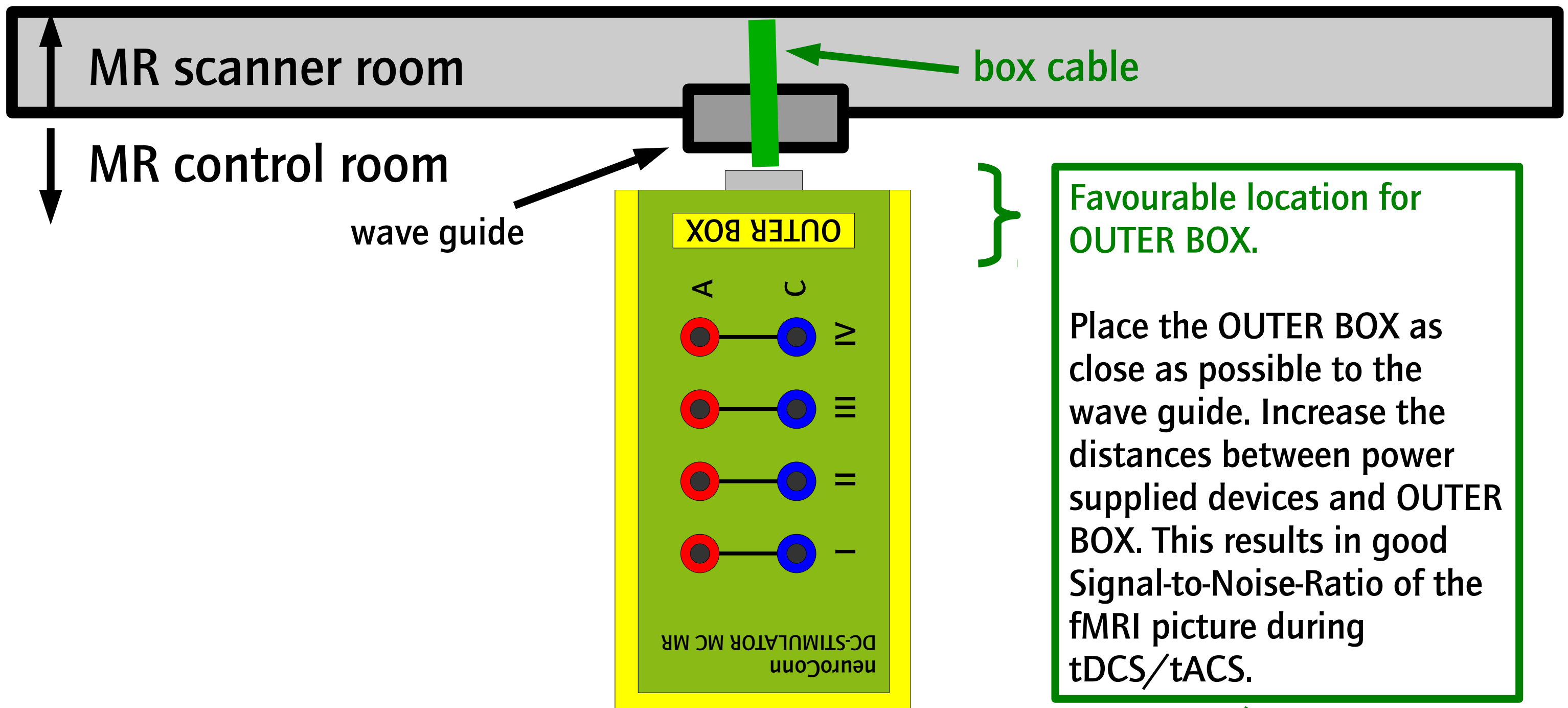

Document version: 1.0 (02/03/16)

Copyright:

All text, images, graphics and multimedia files and their layouts are subject to copyright and any other laws relating intellectual property. They may not be copied for commercial purposes or for transmission, modified or used in any other manner, in particular on other websites.

The content on this document is provided by neuroConn GmbH without assurance or warranty of any kind, whether explicitly or implicitly. All implicit guarantees concerning the merchantability, the suitability for certain purposes or the non-violation of laws and patents are also excluded. Even when we assume that the information that we provide is correct, it may still contain errors or inaccuracies.

Trademarks:

All trademarks mentioned on this website are the legally protected trade marks of the corresponding owners. They are mentioned for information purposes only. This applies in particular to model and brand names and corresponding logos and emblems of the relevant companies.

Legal effect of this disclaimer:

This disclaimer is considered to be part of the document which contained the link to this document. Should parts or individual formulations of this text not correspond to, no longer correspond to or incompletely correspond to the current legal situation, the validity of the content of the remaining parts of the document will be unaffected.

neuroCare Group GmbH
Rindermarkt 7
80331 München
Germany
Phone: +49 89 2154712990

Albert-Einstein-Straße 3
98693 Ilmenau
Germany
Phone: +49 3677 68 979 0

E-Mail: info@neurocaregroup.com
Web: www.neurocaregroup.com

